

139 Caversham Road, Reading, Berkshire, RG1 8AU

Tel: 01189 500 635 Freephone: 0800 731 0727 Fax: 01189 591 167 E-mail: accounts@extinguishers.co.uk Web: www.extinguishers.co.uk

How to use The Transit Chair



locked and rigid.

Unfolding the Chair

1. To unfold the transit chair, stand at the side holding the backrest with one hand and with your other hand push down on the seat frame.
2. When the chair is fully open the seat frame will lock in place.
3. If the chair is fitted with a footrest, this will automatically open out as the chair unfolds.
4. Before using the chair ensure the chair folding mechanism is fully

Folding the chair

1. Standing at the side of the chair, locate the release knob situated on the left side where the backrest meets the seat.
2. Pull the release knob outwards and at the same time lift the seat frame upwards.
3. As the chair folds take the weight of the chair by holding the backrest and continue to fold the chair into its flat state.

Preparing for use

Before using the transit chair ensure the frame is rigid and there are no tears in the upholstery.

If there are brakes fitted to the chair; to apply, press down on the pedal situated at the rear of the caster. To disengage the brake, either press down on the now raised pedal or flip the leading edge of the pedal upwards.

Do's

- Read the instructions carefully before using the transit chair.
- Ensure the chair is cleaned on a regular basis.
- Ensure the chair is inspected on a regular basis.
- Ensure the frame is fully locked before using.
- Ensure the brakes are applied at all times when stationary.

Don'ts

- Do not exceed the safe working load of 115kg (18 stones).
- Do not use if upholstery is excessively worn or torn.
- Do not expose to excessive heat sources or naked flames.
- Do not leave the chair unattended whilst occupied.